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Handgelenk MOBIL SOFT CONTROL

PAD
ARRHYTHMIE
ERKENNUNG

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technology by
microlife

DE

EN

FR

RU

TR

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- ① ON/OFF Button
- ② Display
- ③ Battery Compartment
- ④ Cuff
- ⑤ M button (Memory)
- ⑥ Time Button

Display

- ⑦ Systolic Value
- ⑧ Diastolic Value
- ⑨ Pulse
- ⑩ Date/Time
- ⑪ Traffic Light Display
- ⑫ Stored Value
- ⑬ Heart Arrhythmia Indicator
- ⑭ Pulse Rate
- ⑮ Battery Display

Dear Customer,

Your new **aponorm®** by microlife blood pressure monitor is a reliable medical instrument for taking measurements on the wrist. It is simple to use, accurate and comes highly recommended for blood pressure monitoring in your home. This instrument was developed in collaboration with physicians and clinical tests prove its measurement accuracy to be very high.

EN

Please read through these instructions carefully so that you understand all functions and safety information. We want you to be happy with your **aponorm®** by microlife product. If you have any questions, problems or want to order spare parts, please contact **aponorm®** by microlife-Customer Service. Your pharmacy will be able to give you the address of the **aponorm®** by microlife dealer in your country. Alternatively, visit the Internet at www.aponorm.de where you will find a wealth of invaluable information on our products.

Stay healthy – **aponorm®** by microlife!

Important! Please observe the information in this booklet in order to prevent damage to the blood pressure monitor and avoid error messages.

Protect from moisture

Read the instructions carefully before using this monitor.

Part of the type BF

Table of Contents

1. Important Facts about Blood Pressure and Self-Measurement	19	7. Battery Indicator and Battery change	25
How do I evaluate my blood pressure?		Batteries almost flat	25
2. Using the Instrument for the first Time	20	Batteries flat – replacement	25
Activating the fitted batteries		Which batteries and which procedure?	26
Setting the date and time	21	Using rechargeable batteries	26
3. Taking a Blood Pressure Measurement using this Instrument	21	8. Error Messages	27
4. Appearance of the Heart Arrhythmia Indicator for early Detection	22	9. Safety, Care, Accuracy Test and Disposal	28
		Safety and protection	28
5. Traffic Light Indication in the Display	23	Instrument care	29
		Cleaning the cuff	29
6. Data Memory	23	Accuracy test	29
Viewing the stored values	24	Disposal	29
How not to store a reading	24	10. Guarantee	30
Memory full	24		
Clear all values	24	11. Technical Specifications	31
		Guarantee Card (see Back Cover)	

1. Important Facts about Blood Pressure and Self-Measurement

- **Blood pressure** is the pressure of the blood flowing in the arteries generated by the pumping of the heart. Two values, the **systolic** (upper) value and the **diastolic** (lower) value, are always measured. The instrument also indicates the pulse rate (the number of times the heart beats in a minute).
- **Permanently high blood pressure values can damage your health and must be treated by your doctor!**
- Always discuss your values with your doctor and tell him/her if you have noticed anything unusual or feel unsure. **Never rely on single blood pressure readings.**
- **A number of factors can affect the accuracy of measurements taken on the wrist.** In some cases, the result may differ from the measurement taken on the upper arm. We therefore advise you to compare these values with those produced by the upper arm measurement and discuss them with your doctor.
- Enter your readings in the enclosed **blood pressure diary**. This will give your doctor a quick overview.
- There are many causes of excessively **high blood pressure values**. Your doctor will explain them in more detail and offer treatment where appropriate. Besides medication, the following factors can lower your blood pressure: relaxation techniques, weight loss and exercise.
- **Under no circumstances should you alter the dosages of any drugs prescribed by your doctor!**
- Depending on physical exertion and condition, your blood pressure is subject to wide fluctuations as the day progresses. **You should therefore take your measurements in the same quiet conditions and when you feel relaxed!** Take at least two measurements per day, one in the morning and one in the evening.
- It is quite normal for two measurements taken in quick succession to produce significantly **different results**.
- **Deviations** between measurements taken by your doctor or in the pharmacy and those taken at home are quite normal, as these situations are completely different.

- **Several measurements** provide a much clearer picture than just one single measurement.
- **Leave a small break** of at least 15 seconds between two measurements.
- If you are **pregnant**, you should monitor your blood pressure very closely as it can change drastically during this time!
- If you suffer from an **irregular heartbeat** (arrhythmia, see „Section 4.“), measurements taken with this instrument should only be evaluated after consultation with your doctor.
- **The pulse display is not suitable for checking the frequency of heart pacemakers!**

How do I evaluate my blood pressure?

Table for classifying blood pressure values in adults in accordance with the World Health Organisation (WHO) in 2003. Data in mmHg.

Range	Systolic	Diastolic	Recommendation
Blood pressure too low	<100	<60	Consult your doctor
1. Blood pressure optimum	100 - 120	60 - 80	Self-check
2. Blood pressure normal	120 - 130	80 - 85	Self-check
3. Blood pressure slightly high	130 - 139	85 - 89	Consult your doctor
4. Blood pressure too high	140 - 159	90 - 99	Seek medical advice
5. Blood pressure far too high	160 - 179	100 - 109	Seek medical advice
6. Blood pressure dangerously high	≥180	≥110	Urgently seek medical advice!

The higher value is the one that determines the evaluation.
Example: a readout value between **150/85** or **120/98** mmHg indicates „Blood pressure too high“.

2. Using the Instrument for the First Time

Activating the fitted batteries

Pull out the protective strip projecting from the battery compartment ③.

Setting the date and time

1. **After** the new batteries are fitted, the year number flashes in the display. You can set the year by pressing the M button ⑤. To confirm and then set the month, press the time button ⑥.
2. **You** can now set the month using the M button. Press the time button to confirm and then set the day.
3. **Please** follow the instructions above to set the day, hour and minutes.

4. **Once** you have set the minutes and pressed the time button, the date and time are set and the time is displayed.

5. **If** you want to change the date and time, press and hold the time button down for approx. 3 seconds until the year number starts to flash. Now you can enter the new values as described above.

3. Taking a Blood Pressure Measurement using this Instrument

Checklist for taking a reliable measurement

1. **Avoid** activity, eating or smoking immediately before the measurement.
2. **Sit** down for at least 5 minutes before the measurement – and relax.
3. **Always** take the measurement while in a sitting position on the left or right arm, depending on which arm higher values were measured.
4. **Remove** any items of clothing and your watch, for example, so that your wrist is free.
5. **Always** ensure the cuff is positioned correctly, as shown in the pictures appearing at the beginning of this booklet.
6. **Fit** the cuff comfortably but not too tight. The cuff will cover a wrist diameter of 13.5 to 21.5 cm (5.25 - 8.5 inches).
7. **Support** your arm in a relaxed position and ensure that the instrument is at the same height as your heart.

8. **Press** the ON/OFF button to start the measurement.
9. **The cuff** will now pump up automatically. A reduction of the pumping speed is a normal occurrence. Relax, do not move and do not tense your arm muscles until the measurement result is displayed. Breathe normally and do not talk.*
10. **During** the measurement, the heart symbol ⑭ flashes in the display every time a heartbeat is detected.
11. **The** result, comprising the systolic ⑦ and the diastolic ⑧ blood pressure and the pulse ⑨ is displayed and a longer beep is heard. Note also the explanations on further displays in this booklet.
12. **Remove** and switch off the monitor and enter the result in the enclosed blood pressure pass. (The monitor does switch off automatically after approx. 1 min.).

* You can stop the measurement at any time by pressing the ON/OFF button (e.g. if you feel uneasy or an unpleasant pressure sensation).

4. Appearance of the Heart Arrhythmia Indicator for early Detection

Symbol ⑯ indicates that certain pulse irregularities were detected during the measurement. In this case, the result may deviate from your normal blood pressure – please repeat the measurement. In most cases, this is no cause for concern. However, if the symbol appears on a regular basis (e.g. several times a week with measurements taken daily) we advise you to tell your doctor. Please show your doctor the following explanation:

Information for the doctor on frequent appearance of the Arrhythmia indicator

This instrument is an oscillometric blood pressure monitor that also analyses pulse frequency during measurement.
The instrument is clinically tested.

The arrhythmia symbol is displayed after the measurement if pulse irregularities occur during measurement. If the symbol appears more frequently (e.g. several times per week on

measurements performed daily) we recommend the patient to seek medical advice. The instrument does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

EN

5. Traffic Light Indication in the Display

The bars on the left-hand edge of the traffic light display ⑮ show you the range within which the indicated blood pressure value lies. Depending on the height of the bar, the readout value is either within the normal (green), borderline (yellow) or danger (red) range. The classification corresponds to the 6 ranges in the Table as defined by the WHO, as described in „section 1“.

6. Data Memory

At the end of a measurement, this instrument automatically stores each result, including date and time.

Viewing the stored values

Press the M-button (5) briefly, when the instrument is switched off. The display first shows „M”, and then a value, e.g. „M17”. This means that there are 17 values in the memory. The instrument then switches to the last stored result.

Pressing the M button again displays the previous value. Pressing the M button repeatedly enables you to toggle between one stored value and another.

How not to store a reading

Press the ON/OFF button while the reading is being displayed. Keep the button pressed until „M” is flashing and then release it. Confirm by pressing the M button.

Memory full

☞ Pay attention that the maximum memory capacity of 30 is not exceeded. **When the memory is full, the old values are automatically overwritten with new ones.** Please show the values to a doctor or enter them in a blood pressure pass for evaluation before the memory capacity is reached and the values are overwritten and can no longer be retrieved.

Clear all values

If you are sure that you want to permanently remove all stored values, hold down the M button (the instrument must have been switched off beforehand) until „CL” appears and then release the button. To permanently clear the memory, press the M button while „CL” is flashing. Individual values cannot be cleared .

7. Battery Indicator and Battery change

Batteries almost flat

When the batteries are approximately 3/4 used the battery symbol ⑯ will flash as soon as the instrument is switched on (partly filled battery displayed). Although the instrument will continue to measure reliably, you should obtain replacement batteries.

Batteries flat – replacement

When the batteries are flat, the battery symbol ⑯ will flash as soon as the instrument is switched on (flat battery displayed). You cannot take any further measurements and must replace the batteries.

1. Open the battery compartment ③ by sliding the cover away in the direction of the arrow.
2. Replace the batteries – ensure correct polarity as shown by the symbols on the battery cover.
3. To set date and time, follow the procedure described in „section 2.“
 - The memory retains all values, although date and time must be reset – the year number therefore flashes automatically after the batteries have been replaced.

EN

Which batteries and which procedure?

- Please use 2 new, long-life 1.5 V, size AAA alkaline batteries.
Use only batteries of one brand at a time.
- Do not use batteries beyond their date of expiry.
- Remove batteries if the instrument is not going to be used for a prolonged period.

Using rechargeable batteries

- You can also operate this instrument using rechargeable batteries.
- Please use only NiMH type reusable batteries!
- The batteries must be removed and recharged if the battery symbol (battery flat) appears! They must not remain inside the instrument, as they may become damaged (total discharge as a result of low use of the instrument, even when switched off).
- Always remove the rechargeable batteries if you do not intend to use the instrument for a week or more!
- The batteries cannot be charged in the blood pressure monitor!
Recharge these batteries in an external charger and observe the information regarding charging, care and durability!

8. Error Messages

If an error occurs during the measurement, the measurement is interrupted and an error message, e.g. „ERR 3“ is displayed.

Error	Designation	Potential cause and remedy
„ERR 1“	Signal too weak	The pulse signals on the cuff are too weak. Re-position the cuff and repeat the measurement.*
„ERR 2“	Error signal	During the measurement, error signals were detected by the cuff, caused for instance by movement or muscle tension. Repeat the measurement, keeping your arm still.
„ERR 3“	No pressure in the cuff	An adequate pressure cannot be generated in the cuff. A leak may have occurred. Check that the cuff is correctly connected and is not too loose. Replace the batteries if necessary. Repeat the measurement.
„ERR 5“	Abnormal result	The measuring signals are inaccurate and no result can therefore be displayed. Read through the checklist for performing reliable measurements and then repeat the measurement.*

Error	Designation	Potential cause and remedy
„HI“	Pulse or cuff pressure too high	The pressure in the cuff is too high (over 300 mmHg) OR the pulse is too high (over 200 beats per minute). Relax for 5 minutes and repeat the measurement.*
„LO“	Pulse too low	The pulse is too low (less than 40 beats per minute). Repeat the measurement.*

* Please consult your doctor if this or any other problem occurs repeatedly.

If you think the results are unusual, please read through the information in „section 1.“ carefully.

9. Safety, Care, Accuracy Test and Disposal

Safety and protection

This instrument may be used only for the purpose described in this booklet. The manufacturer cannot be held liable for damage caused by incorrect application

- **This** instrument comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the „Technical specifications“ section!

- **Protect** it from:

- water and moisture
- extreme temperatures
- impact and dropping
- contamination and dust
- direct sunlight
- heat and cold

- The cuffs are sensitive and must be handled with care.
- Only pump up the cuff when fitted.
- Do not use this device close to strong electromagnetic fields such as mobile telephones or radio installations, as they can impair its function. We recommend observing a minimum distance of 1 metre. If you cannot maintain the minimum distance please check the proper function of the device before use.
- Do not use the instrument if you think it is damaged or if you notice anything unusual.
- Never open the instrument.
- If the instrument is not going to be used for a prolonged period the batteries should be removed.
- Read the further safety instructions in the individual sections of this booklet.

Ensure that children do not use the instrument unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is fitted with cables or tubes.

Instrument care

Clean the instrument only with a soft, dry cloth.

Cleaning the cuff

Carefully remove any marks on the cuff with a damp cloth and mild detergent.

Accuracy test

We recommend this instrument is tested for accuracy every 2 years or after mechanical impact (e.g. being dropped). Please contact the **aponorm®** by microlife service to arrange the test (see foreword).

Disposal

Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not as domestic waste.

10. Guarantee

This instrument is covered by a **5 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the dealer (see back) confirming date of purchase or the till receipt.

- The guarantee covers both instrument and cuff. Batteries are not included.
- Opening or altering the instrument invalidates the guarantee.
- The guarantee does not cover damage caused by improper handling, discharged batteries, accidents or failure to observe the operating instructions.

Please contact the **aponorm®** by microlife service (see foreword).

11. Technical Specifications

Operating conditions: 10 - 40 °C / 50 - 104 °F

Storage conditions:

-20 - +55 °C / -4 - +131 °F

15 - 90% relative maximum humidity

Weight: 130 g (including batteries)

Dimensions: 80 x 70 x 70 mm

Measuring procedure:

oscillometric, corresponding to Korotkoff method:
phase I systolic, phase V diastolic

Measurement range:

20 - 280 mmHg – blood pressure

40 - 200 beats per minute – pulse

Cuff pressure display range: 0 - 299 mmHg

Resolution: 1 mmHg

Static accuracy: pressure within \pm 3 mmHg

Pulse accuracy: \pm 5 % of the readout value

Voltage source: 2 x 1.5 V Alkaline-Batteries; size AAA

Battery life: approx. 320 measurements (new batteries)

IP class: IP 20

Reference to standards:

EN 1060-1 /-3 /-4; IEC 60601-1-2 (EMC); IEC 60601-1-11

Expected service life:

Device: 5 years or 10,000 measurements; accessories: 2 years

EN

This device complies with the requirements of the Medical
Device Directive 93/42/EEC.

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Microlife AG
WEPA Apothekenbedarf GmbH & Co KG