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technology by  
*microlife*

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## Stirn-Thermometer contact-free

DE

EN

FR

RU

Clinically Tested  
in Europe

# Garantiekarte

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# Introduction

**Dear customer,**  
**Congratulations on your purchase of the aponorm® by microlife forehead thermometer contact-free.**

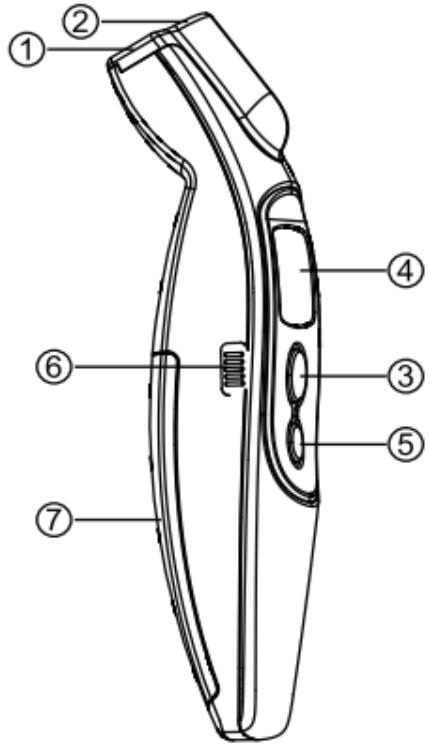
This thermometer is a quality device for temperature measurement. Thanks to its unique technology the thermometer enables stable readings for all measurements. Non-contact measurement facilitates monitoring of the body temperature particularly of infants.

These operating instructions describe all functions of your device. They include important safety information and

instructions for the use of the thermometer. **Please read all information in the operating instructions thoroughly before using the device.** Please retain these instructions for further reference. If you have any questions or problems, please contact your local pharmacy. Your pharmacist is the person to contact for all questions concerning the thermometer. For more detailed information on our products please refer to [www.aponorm.de](http://www.aponorm.de).

**We wish you good health at all times!**

**aponorm® – Your pharmacy brand**

- |   |  |   |  |   |  |
|---|--|---|--|---|--|
| ⑧ |  | ⑯ |  | ⑳ |  |
| ⑨ |  | ⑯ |  | ㉑ |  |
| ⑩ |  | ⑯ |  | ㉒ |  |
| ⑪ |  | ⑯ |  | ㉓ |  |
| ⑫ |  | ⑯ |  | ㉔ |  |
| ⑬ |  | ⑯ |  | ㉕ |  |

- ① Measuring Sensor
- ② Tracking light
- ③ START Button
- ④ Display
- ⑤ ON/OFF Button
- ⑥ Mode Switch
- ⑦ Battery Compartment Cover
- ⑧ All Segments displayed
- ⑨ Memory
- ⑩ Ready for Measurement
- ⑪ Measurement complete
- ⑫ Body Mode
- ⑬ Object Mode
- ⑭ Low Battery Indication
- ⑮ Changing between Celsius and Fahrenheit
- ⑯ Recall Mode
- ⑰ Recall the last 30 Readings
- ⑱ Measured Temperature too high
- ⑲ Measured Temperature too low
- ⑳ Ambient Temperature too high
- ㉑ Ambient Temperature too low
- ㉒ Error Function Display
- ㉓ Blank Display
- ㉔ Flat Battery
- ㉕ Replacing the Battery

Read the instructions carefully before using this device.

Type BF applied part

# Table of Contents

## 1.

- |     |                                     |       |
|-----|-------------------------------------|-------|
| 1.1 | Safety information and instructions | S. 42 |
| 1.2 | Scope of delivery, package          | S. 44 |
| 1.3 | Benefits                            | S. 45 |

## 2.

- |     |   |       |
|-----|---|-------|
| 2.1 | Display symbols                                       | S. 47 |
| 2.2 | Switching between the body and the object modes       | S. 48 |
| 2.3 | Use   | S. 48 |
| 2.4 | Switch between degrees Celsius and degrees Fahrenheit | S. 52 |
| 2.5 | Retrieving the last measuring values stored           | S. 52 |
| 2.6 | Error messages and problems                           | S. 54 |

**3.**

- |                               |       |
|-------------------------------|-------|
| 3.1 Cleaning and disinfection | S. 55 |
| 3.2 Battery change            | S. 56 |

**4.**

- |                    |       |
|--------------------|-------|
| 4.1 Technical data | S. 58 |
|--------------------|-------|

## 1.1 Safety information and instructions

### Safety

**This unit is designed for private use only.**

- The patient and the thermometer must be in the same constant-temperature environment for at least 30 minutes before a measurement can be made. This instrument may be used only for the purpose described in these instructions. The manufacturer will not accept any liability for damage caused by improper use.
- **Do not immerse the device in water or other liquids. For cleaning and disinfection please observe the instructions in section „Cleaning and disinfection“.**

- Do not use the thermometer if you think it is damaged or notice anything unusual.
- Do not open the thermometer housing. This is only permissible for replacing the batteries (see instructions).
- The temperature of the forehead and temples is different from the **body's core temperature (rectal/oral)**. Often patients' reaction to illness is sweat formation on the forehead. This is what we call the „cool skin effect“. In this case temperature measurements will be lower than expected. Please repeat the measurements every 15 minutes.
- This instrument comprises sensitive components and must be handled with care. Observe the storage and operating

conditions described in the „Technical data“ section.

- In case of malfunction or damage do not repair the device yourself. This will void all warranty claims. Have repairs performed only by authorized service partners. Ask your pharmacist for the address.
- Do not let children use the thermometer without supervision. Some of the components are so small that children can swallow them or choke on them.
- Do not use the thermometer in a moist environment.
- If the instrument is not going to be used for a prolonged period, the batteries should be removed.

**The device must not be exposed to:**

- Extreme temperatures and/or temperature variations
- Impact and dropping
- Contamination and dust
- Strong solar irradiation

**WARNING:**

Using the device is no alternative for consulting a doctor.

## 1.2 Scope of delivery, package

**Please check first that the thermometer is complete and that there is no damage. If in doubt do not commission the device, but return it to your pharmacy.**

Scope of supply:

- 1 aponorm® forehead thermometer contact-free
- 2 x 1.5 V batteries (type LR03, AAA) (already fitted!)
- 1 storage dish and pouch
- 1 set of instructions for use

All packages are recyclable and can be fed back into the raw materials cycle. Please ensure proper disposal of packaging material no longer required. If you detect transport damage on unpacking your thermometer, please contact your pharmacy immediately.

### **WARNING:**

Do not let children handle or play with the packaging film. Risk of suffocation!

## 1.3 Benefits

EN

### Quick measurement

The innovative infrared technology enables measurements without the thermometer even touching the object or skin. When handled properly, the thermometer allows to make reliable and hygienic measurements within just seconds.

### Measurement of body and surface temperatures possible

The thermometer has an expanded measuring range from 0 °C - 100.0 °C (32.0 °F - 212.0 °F). This means that it can be used for **measuring body temperature** (body mode) as well as for **measuring surface temperatures** (object mode), e. g. of the following objects:

- surface temperature of the milk in baby bottles
- surface temperature of baby bath water
- ambient temperature

**CAUTION:** The surface temperature of an object differs from its core temperature. For measuring the core temperature of liquids use a suitable waterproof thermometer.

### Accurate and reliable

The unique measuring probe with modern infrared sensor ensures that all measurements are accurate and reliable when the thermometer is used properly.

## **Automatic memory display**

When the thermometer is switched on, the last measuring value is displayed for 2 seconds.

## **Convenient and simple handling**

- The ergonomic design allows convenient and simple thermometer handling.
- You can even measure the temperature of a sleeping child without disturbing it.
- The thermometer is very fast and therefore convenient particularly for children or bedridden patients.

## **Retrieval of several measuring values**

The last 30 measuring values are stored and can be retrieved in the memory mode. This helps to better follow temperature variations, e. g. a fever curve.

## **Fever warning**

10 short acoustic signals and red background lighting indicate a patient temperature exceeding 37.4 °C (99.3 °F).

## **Reliable and hygienic**

- No direct contact with the skin
- Suitable for children

## 2.1 Display symbols

EN

### List of all display symbols:

Press the ON/OFF button ⑤ to switch the thermometer on.  
All display symbols are visible for a period of 2 seconds.

### Memory:

The last measuring value lights up automatically on the display for 2 seconds.

### Ready for measurement:

The thermometer is ready for measurement when the «°C» or «°F» symbol flashes while the mode symbol (body or object) is displayed permanently.

### Measurement finished:

The measuring value appears on the display with the «°C» or «°F» symbol and the mode symbol. When the «°C» or «°F» symbol flashes again, the thermometer is ready for another measurement.

### Battery low:

When the thermometer is switched on, the battery symbol «▼» lights up to remind the user that the battery must be replaced.

## 2.2 Switching between the body and the object modes/2.3 Use

### Switching between the body and the object modes

In order to change from the body mode to the object mode, push the mode switch ⑥ on the side of the thermometer down. To return to the body mode, push the switch up again.

### Use

#### Temperature measurement in the body mode

1. Press the ON/OFF button ⑤. The display lights up and shows all display symbols for a period of 2 seconds.  
**NOTE:** The value of the last measurement is displayed automatically for 2 seconds along with the symbol «M».
2. The thermometer is ready for measurement as soon as the «°C» or «°F» symbol flashes and an acoustic signal is heard.
3. Point the thermometer to the middle of the patient's forehead at a distance of 1 to 5 cm from the forehead.

If the forehead is covered with hair or in sweat or dirt, brush aside the hair and clean the forehead first in order to ensure accurate measurement.

4. Press the START button ③ and move the thermometer evenly from the middle of the forehead to the temples (approx. 1 cm above the eyebrow). The blue indicator light indicates the measuring range. After 3 seconds a long acoustic signal is heard, which confirms the end of the measuring operation.

NOTE: This acoustic signal can neither be changed nor switched off

If you did not reach the temples before the acoustic signal sounded, repeat the measurements as described on pages 48/49, but move the thermometer a bit faster.

5. Read the temperature measured on the display.

### Temperature measurement in the object mode

1. Follow steps 1 to 3 as described on page 48. Point the thermometer to the centre of the object the temperature of which you want to measure, at a distance of 1 to 5 cm from its surface. Press the START button ③. After 3 seconds a long acoustic signal is heard, which confirms the end of the measuring operation.

2. Read the temperature measured on the display.

## NOTE:

- Do not measure the temperature during or immediately after breastfeeding a baby.
- Do not use the thermometer in an environment with high atmospheric humidity.
- Before or during the measurement the person to be measured must not drink, eat or take exercise. During measuring the patient must lie still. Movements of the head can affect the measuring result.

- Do not take the thermometer from the measuring area (body or object) before the acoustic end signal is heard.
- Although the thermometer is not in contact with the patient's skin, clean the probe thoroughly with an alcohol swab and wait for 15 minutes before measuring another person's temperature. Due to external influences the lens can become dust-covered.
- 10 short acoustic signals and red background lighting indicate that the patient's temperature exceeds 37.4 °C (99.3 °F).
- The **aponorm®** forehead thermometer contact-free can be used on infants older than 18 months. Nevertheless, to be on the safe side, we recommend checking the temperature with a conventional medical thermometer!

The evolution of the disease is best checked by taking the patient's temperature 2 to 3 times a day at intervals of 4 to 6 hours. Important: Every time the measurement must be made in the same place and on the same body part!

**NOTE:**

- In the following cases we recommend a check measurement with a conventional medical thermometer:
  1. When you are becoming familiar with the thermometer it may be useful to make several measurements until you get stable and unchanging measuring values.

2. Children younger than three years and having a weak immune system for whom it is decisive to know whether they are running a temperature or not. For children showing alarming symptoms in case of fever (e. g. fever convulsions).
  3. If the value measured is lower than expected.
- Measuring results taken on different body parts cannot be compared, as the normal human body temperature varies, depending on the place of measurement and time of the day. It is highest in the evening and lowest about one hour before waking up.

## 2.4 Switch between degrees Celsius and degrees Fahrenheit/2.5 Retrieving the last measuring values

- Normal body temperature ranges:
  - forehead (at a distance of 1 to 5 cm):  
35.4 °C - 37.4 °C / 95.7 °F - 99.3 °F
  - axillary (under the armpit):  
34.7 °C - 37.7 °C / 94.5 °F – 99.1 °F
  - oral (under the tongue): 35.5°C - 37.5 °C / 95.5 °F – 99.5 °F
  - rectal (in the anus): 36.6 °C - 38.0 °C / 97.9 °F - 100.4 °F

### Switch between degrees Celsius and degrees Fahrenheit

The thermometer can show the temperature measured either in degrees Celsius (°C) or in degrees Fahrenheit (°F). To change the display from °C over to °F, switch the device

off «» and press the START button ③ for 5 seconds. After these 5 seconds the temperature unit set at present («°C» or «°F» symbol) is shown on the display.

**Now switch** the temperature unit between °C and °F by pressing the START button ③. Once you selected the unit desired, wait for about 5 seconds. The thermometer will automatically change into the „**Ready for measurement**“ mode.

### Retrieving the last measuring values stored

The thermometer automatically stores the last 30 values measured. They values can be retrieved from the memory.

- **Memory mode:** To enter the memory mode, press the START button ③ with the thermometer switched off. The memory symbol «M» flashes.
- **Retrieval 1 - the last measurement:** To display the value measured last, press the START button ③. The display shows «1» and the memory symbol «M». Then the stored value is shown.
- **Retrieving the 30 last measuring values stored:** Press the START button ③ and hold it down to display the last 30 measuring values one by one. The value measured last (the latest value) is in position 1. The value measured first (the oldest value) is in position 30. When you press the START button ③ again after you have viewed the

30 measurement values, this sequence will re-start, i. e. with measuring value 1.

EN

## Erasing the memory

1. Press the START button ③ and the ON/OFF button ⑤ simultaneously. Release the ON/OFF button ⑤ while holding the START button ③ down.
2. As soon as the last measurement is shown (indicated by the «M» at the right-hand edge) press START ③ twice. The display flashes twice. A beep indicates that the memory has been erased.

## 2.6 Error messages and problems

- **Temperatures measured too high:**

**In body mode:** Display «H», the temperature measured exceeds 42.2 °C (108.0 °F).

**In object mode:** Display «H», the temperature measured exceeds 100 °C (212 °F).

- **Temperatures measured too low:**

**In body mode:** Display «L», the temperature measured is less than 34.0 °C (93.2 °F).

**In object mode:** Display «L», the temperature measured is less than 0 °C (32 °F).

- **Ambient temperature too high:**

At excessive ambient temperatures, no measurement is possible. Display «H» in combination with «». The ambient temperature exceeds 40.0 °C (104.0 °F), independently of the measuring mode you are in.

- **Ambient temperature too low:**

At too low ambient temperatures, no measurement is possible. Display «L» in combination with «». The ambient temperature is less than 16.0 °C (60.8 °F) in the body mode or 5.0 °C (41.0 °F) in the object mode.

### 3.1. Cleaning and disinfection

- **Malfunction signalization:**

There is a malfunction or failure in the system.

- **Empty display:**

Check whether the batteries have been inserted correctly.

Check the battery polarity (<+> and <->).

- **Battery empty:**

Replace the batteries without delay if the «▼» symbol is the only symbol lit.

#### Cleaning and disinfection

Use an alcohol sponge or a cotton cloth moistened with alcohol (70 % isopropanol) to clean the thermometer housing and the measuring probe. Please make sure that no liquid enters the thermometer. Do not use aggressive cleaners, solvents or benzene for cleaning. Do not immerse the thermometer in water or other cleaning liquids. Take care not to scratch the surface of the measuring probe and of the display.

EN

## 3.2 Battery change

The thermometer is supplied with two new long-life 1.5 V batteries AAA. The batteries must be replaced if «▼» ⑩ is the only symbol shown on the display. Use a screwdriver to open the battery compartment cover ②⁵. Replace the batteries – ensure correct polarity as shown by the symbols in the compartment.

### WARNING

#### Battery safety information and instructions

- Do not disassemble the batteries!
- If necessary, clean the battery and thermometer contacts before inserting the batteries.
- Remove empty batteries from the thermometer immediately.
- Increased risk of leakage. Avoid contact with skin, eyes and mucous membranes. If you come into contact with battery acid, immediately rinse the affected area with large amounts of water and immediately consult a doctor.

- If a person swallows a battery, consult a doctor without delay!
- Replace all batteries at the same time.
- Insert only one type of batteries. Do not mix different battery types or new and used batteries.
- Insert the batteries correctly, observe battery polarity.
- Keep the battery compartment closed at all times.
- Remove the batteries if the thermometer is not to be used for a rather long period of time.
- Keep the batteries out of reach of children.
- Do not recharge the batteries. Explosion hazard!
- Do not short-circuit. Explosion hazard!
- Do not throw the thermometer into open fire. Explosion hazard!
- Store new batteries in their package and at a distance from metallic objects to prevent a short-circuit.
- Do not dispose of used batteries and accumulators with domestic waste, but with hazardous waste or turn them in at the battery collecting point of a retail outlet.

## 4.1 Technical data

**Type:** aponorm® forehead thermometer contact-free

**Measuring range:**

Body mode: 34.0 °C – 42.2 °C (93.2 °F – 108.0 °F)

Object mode: 0 °C – 100.0 °C (32 °F - 212.0 °F)

**Measuring resolution:** 0.1 °C / °F

**Measuring accuracy:**

Laboratory:  $\pm 0.2$  °C, 36.0 °C – 39.0 °C

(The measuring accuracy is 0.2 °C for temperatures between 36.0 °C and 39.0 °C) ( $\pm 0.4$  °F, from 96.8 °F to 102.2 °F)

**Display:** Liquid-crystal display (LCD)

**Acoustic alarms:**

- The thermometer is switched on and ready for measurement: 1 short acoustic signal.
- Conclusion of measurement: 1 long acoustic signal (1 s) if the value measured is below 37.5 °C (99.4 °F), 10 short acoustic signals if the value measured is equal to or exceeds 37.5 °C (99.4 °F).
- System error or malfunction: 3 short beeps

**Memory:**

- The value measured last is shown.
- 30 measurements can be retrieved in the memory mode.

**Background lighting:**

- The display lights up green for 4 seconds when the thermometer is switched on.
- The display lights up green for 5 seconds if the temperature measured is less than 37.5 °C (99.4 °F).
- The display lights up red for 5 seconds if the temperature measured is equal to or exceeds 37.5 °C (99.4 °F).

**Operating temperature:**

Body mode: 16 °C – 40.0 °C (60.8 °F – 104.0 °F)

Object mode: 5 °C – 40.0 °C (41 °F – 104.0 °F)

**Storage temperature:**

-20 °C to +50 °C (-4 °F to 122 °F)  
15 – 95 % relative humidity

**Automatic switch-off:**

Approx. 1 minute after the last measurement

**Batteries:** 2 x 1.5 V batteries; size AAA

**Dimensions:** 150 x 40 x 39 mm

**Weight:** 81 g (including batteries), 56 g (without batteries)

**Reference to standards:**

ASTM E1965; IEC 60601-1; IEC 60601-1-2 (EMC)

## **Warranty**

This thermometer is covered by a **5 year guarantee** from the date of purchase. The guarantee is valid only on presentation of the guarantee card filled in by the pharmacy (see last page) confirming the date of purchase or the till receipt.

The manufacturer will not reimburse the costs for return of the thermometer by customers outside Germany or Austria. Warranty claims submitted by customers outside Germany or Austria can only be handled if these customers pay in advance the cost of the return shipment of the thermometer to them. Any statutory warranty provisions shall remain unaffected.

- The warranty covers only the thermometer. Batteries and package are not covered by the warranty.
- Opening of or modification to the thermometer by the user voids all warranty claims.
- The warranty does not cover damage caused by improper use, battery leakage, accidents or failure to observe the instructions for use.

**Warranty  
5 years**

**Notes:**

- Subject to technical changes without notice.
- It may be the case that the device does not work if the temperature or voltage does not correspond to the values indicated in the technical data section.

**Disposal**

Electronic devices must not be disposed of as normal domestic waste, but they must be disposed of in accordance with the local provisions valid.

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WEPA Apothekenbedarf GmbH & Co. KG
Microlife AG