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# Ohr- Thermometer

COMFORT 3

DE

EN

TR

RU

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**1sec**

technology by  
**microlife<sup>®</sup>**

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**Dear customer,**

**Congratulations on purchasing this aponorm®  
by microlife ear thermometer COMFORT 3.**

This thermometer is a quality device for regularly checking and measuring temperature. It is clinically tested and conforms to international standards. Every time the thermometer is switched on, it performs a self-test in order to ensure the specified measuring accuracy at all times.

Thanks to its unique technology the thermometer enables stable readings for each measurement. Fast measurement (just 1 second) facilitates monitoring of the body temperature particularly for infants.

These instructions describe all functions of your thermometer. They include important safety information and instructions for the use of the thermometer. **Please read all information**

WEPA Apothekenbedarf GmbH & Co. KG, 56204 Hillscheid, [www.aponorm.de](http://www.aponorm.de)

**in the operating instructions thoroughly before using the thermometer.** Please retain the warranty card and these instructions for further reference.

If you have any questions or problems, please contact your local pharmacy. Your pharmacist is the person to speak to for all questions concerning the thermometer. For more detailed information on our products please refer to **[www.aponorm.de](http://www.aponorm.de)**.

**We wish you good health at all times!**

**aponorm®** – Your pharmacy brand

Read the instructions carefully before using this device.


Type BF applied part

Important! Please observe the information in this booklet in order to prevent damages to the blood pressure monitor and avoid error messages.

EN

## The thermometer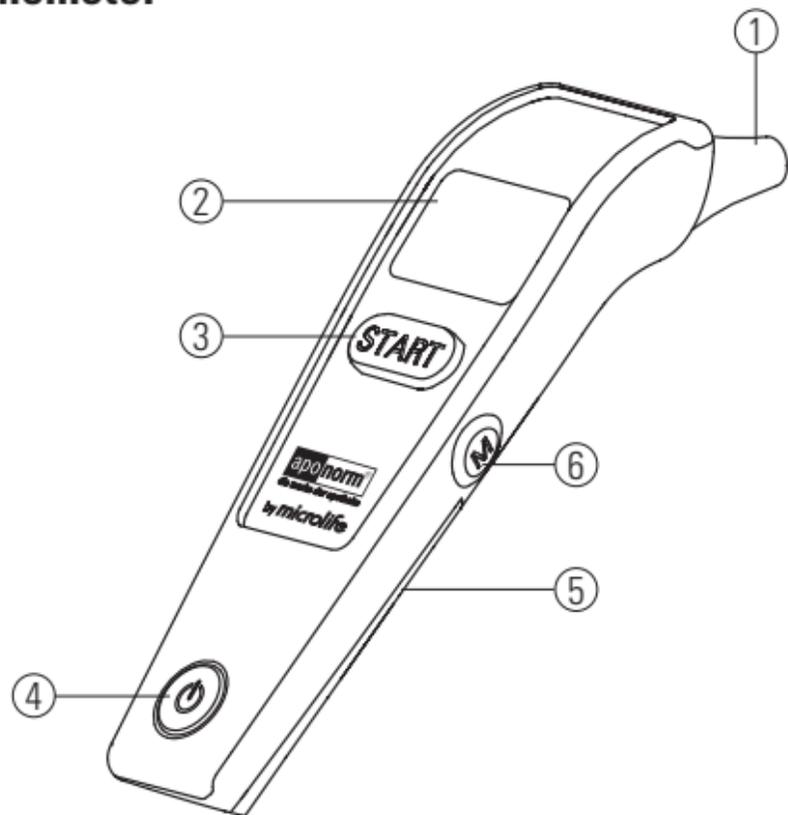

### Thermometer:

- ① Measuring probe
- ② Display
- ③ START button
- ④ On/Off switch
- ⑤ Battery compartment cover
- ⑥ M button

# Table of Contents

## 1. 1. Before use

1.1 Safety information and instructions	S. 38
1.2 Contents and packaging	S. 40
1.3 Thermometer features	S. 41

## 2. During use

2.1 Display symbols	S. 43
2.2 Setting of time and date	S. 46
2.3 Use	S. 50
2.4 Retrieving the 30 readings stored	S. 55
2.5 Error messages and troubleshooting	S. 57

## 3. After use

3.1 Cleaning and disinfection	S. 59
3.2 Battery change	S. 60

## 4. Technical data

S. 62
-------

## 1.1 Safety information and instructions

### Safety

**This unit is designed for private use only.**

- This thermometer must only be used for the purposes described in these instructions. The manufacturer will not accept any liability for damage caused by improper use.
- **The thermometer is not waterproof. Do not immerse it in water or other liquids. For cleaning and disinfection please observe the instructions in section “Cleaning and disinfection”.**
- Do not use this thermometer if you think it is damaged or if you notice anything unusual.

- Open the battery compartment cover only for battery replacement (see instructions).
- **Earwax in the ear canal can result in lower temperature readings. Therefore ensure that the ear canal is clean.**
- **Clean** the measuring probe **after each measurement** as indicated in the instructions. Contamination of the lens can also result in lower temperature readings.
- If the reading measured does not correspond to the patient's condition, please repeat the measurement every 15 minutes. This helps to faster recognize temperature variations, eg during the course of a fever.

Keep dry.

- This thermometer contains sensitive components and must be handled carefully. Observe the storage and operating conditions described in section “4. Technical data”.
  - In case of malfunction or damage do not repair the thermometer yourself. This will void all warranty claims. Please contact the pharmacy where you bought the device. Your pharmacist will help you with all questions concerning the thermometer.
  - The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.
- Please ensure that unsupervised children do not use this thermometer; some parts are small enough to be swallowed.
  - Do not use the thermometer in a humid environment.
  - If the thermometer is not going to be used for a prolonged period the batteries should be removed.
  - **Protect the thermometer from:**
    - extreme temperatures and/or temperature variations
    - impact and falls
    - contamination and dust
    - direct sunlight

**WARNING:**

Using the thermometer is not an alternative to consulting a physician.

## 1.2 Contents and packaging

**Please check first that the thermometer is complete and not damaged. If in doubt do not use the thermometer and return it to your pharmacy.**

Contents:

- 1 **aponorm**<sup>®</sup> ear thermometer COMFORT 3
- 1 x battery CR 2032 (button cell already inserted)
- 1 storage case
- 1 set of instructions for use

All packaging is recyclable and can be fed back into the raw materials cycle. Please ensure proper disposal of any packaging material no longer required. If you find any transport damage on unpacking your thermometer, please contact your pharmacy immediately.

**WARNING:**

Keep packaging film away from babies and young children. Risk of suffocation!

## 1.3 Thermometer features

### Measuring of body and surface temperatures

The innovative infrared technology allows measuring the ear temperature within one second.

### Multiple uses (extended measuring range)

The thermometer has an expanded measuring range from 0 °C - 100 °C (32.0 °F - 212.0 °F). This means that it can be used for measuring the body temperature as well as for measuring surface temperatures of the following objects:

- Surface temperature of milk in baby feeding bottles
- Ambient temperature

### Without protective sheath

The thermometer is user-friendly and cost-saving, as no protective sheaths are required for temperature measuring. However, the thermometer must be cleaned immediately after each use. Otherwise correct readings **cannot** be ensured..

EN

### Measuring probe LED

The thermometer is equipped with an LED light on the measuring probe in order to facilitate pointing at the measuring point in the dark.

### Accurate and reliable

The unique measuring probe with modern infrared sensor ensures that all readings are accurate and reliable.

### **Convenient and simple handling**

- The ergonomic design allows convenient and simple thermometer handling.
- You can even measure the temperature of a sleeping child without disturbing them.
- The thermometer is very fast and therefore convenient, particularly for children.

### **Retrieval of several readings**

In the memory mode, the user can retrieve the last 30 readings along with their date and time. This helps to better follow temperature variations.

### **Reliable and hygienic**

- No risk of breakage of glass, no hazard of mercury absorption.
- Absolutely safe for use with children.
- When you clean the measuring probe with a cotton cloth moistened with alcohol, the use of the thermometer is hygienic for the entire family.
- After use, the cleaning prompt "CLEAN ME" appears on the display.


### **Fever warning**

10 short beeps and a red LCD background lighting indicate a patient temperature equal to or exceeding 37.5 °C (99.5 °F).

## 2.1 Display symbols


### Display of all segments:

Press the ON/OFF button (4) to switch the thermometer on:  
All segments are visible for a period of 1 second.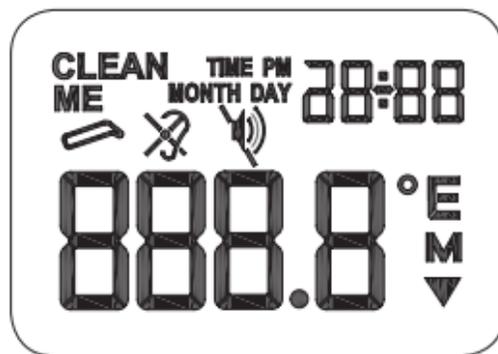

### Ready for use:

The thermometer is ready for use, and the «°C» or «°F» symbol flashes. The LED light on the measuring probe lights up for about 10 seconds.

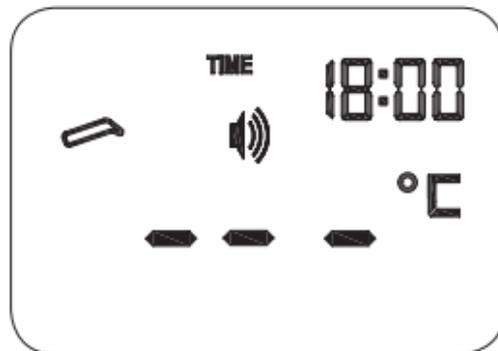
### Measurement finished:

The reading appears on display ② with the «°C» or «°F» symbol. When the «°C» or «°F» symbol flashes again, the thermometer is ready for taking another reading.


### Display of "not ear" temperature reading:

The symbol of a crossed-out ear appears on display ②, when the reading is outside the temperature range from 32,0 to 42,2 °C (89.6 to 107.9 °F).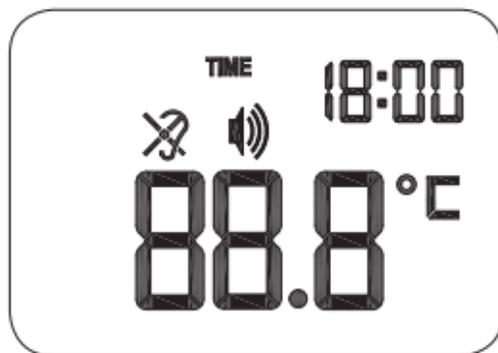

**Battery low:**

When the thermometer is switched on, the battery symbol «▼» lights up permanently to remind the user that the battery must be replaced.

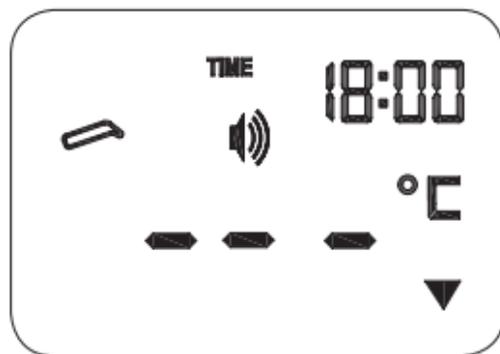## 2.2 Setting of time and date

1. When using the thermometer for the first time or when new batteries have been inserted, the year flashes in the display. Press the START button ③ to set the correct year. Press the M button ⑥ for confirmation and for changing to the month setting mode.
2. Now the correct month can be set by pressing the START button ③. Press the M button ⑥ for confirmation and for changing to the date setting mode.

3. Follow the instructions above to set the date, choose the 12 or 24 hour clock and set the hours and minutes.

4. Once you have set the minutes and pressed the M button ⑥, the date and time are set, and the time is displayed.

If you do not press any other button within 20 seconds, the thermometer will automatically revert to the “Ready for use” mode.

**To interrupt the time setting:**

Press the ON/OFF button ④ when setting the time. The display will show the date/time as "--:--". Then press the ON/OFF button ④ in order to start temperature measurement. If you do not press any other button within 60 seconds, the thermometer will automatically switch off.


**Changing the present date and time settings:**

Keep the M button ⑥ pressed (when the thermometer is switched off) for approx. 3 seconds until the year indication flashes. Now you can enter the new values as described above.

### Setting the acoustic signal

1. When the thermometer is switched off, keep the ON/OFF button ④ pressed for about 5 seconds in order to set the acoustic signal.
2. Press the ON/OFF button ④ again in order to switch the acoustic signal either on or off. This acoustic signal is activated if the acoustic signal icon is not crossed out.

If you do not press any other button within 5 seconds, the thermometer will automatically revert to the "Ready for use" mode.
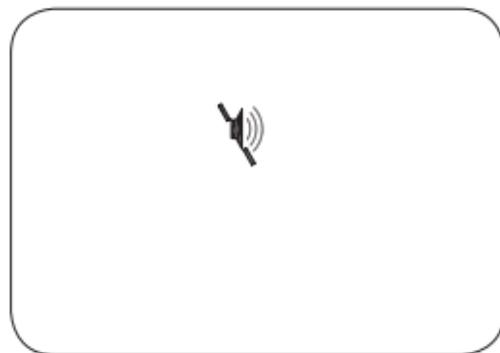


## 2.3 Use

The thermometer measures the infrared energy emitted by the middle ear and adjacent tissue. This energy is captured by lenses and converted into temperature values. The readings obtained directly from the middle ear (eardrum) are the most accurate. Measurements on the adjacent tissue in the ear canal yield lower readings, which can result in an incorrect fever diagnosis.

### Starting the measurement

1. Press the ON/OFF button ④. Display ② lights up and shows all segments for a period of 1 second.
2. The thermometer is ready for use as soon as the «°C» or “°F” symbol flashes and a beep is heard.

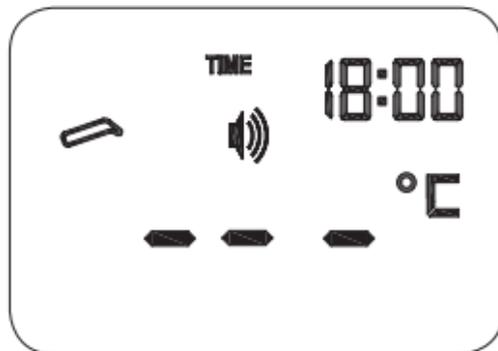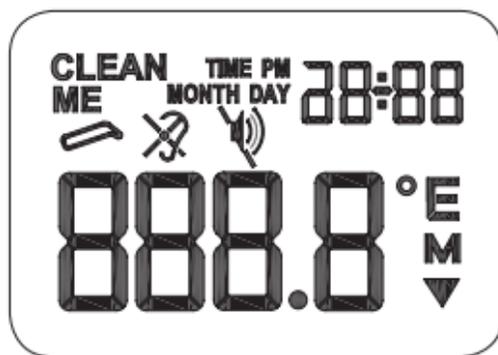3. The LED light of the measuring probe lights up and goes out automatically after 10 seconds.
4. Straighten the ear canal by gently pulling the ear upwards and/or to the rear in order to give the probe an unimpeded "view" of the ear drum.
  - Children younger than 1 year: Gently pull the ear to the rear.
  - Children from 1 year and grown-ups: Gently pull the ear upwards and to the rear.
5. While you gently pull the ear, insert the measuring probe into the ear canal (**about 1 second**) and **immediately** press the START button ③. Release the button and wait for the acoustic signal. This signal confirms the end of the measurement.
6. Pull the thermometer from the ear canal. The display shows the temperature reading.

**Important information about the acoustic signal:** This acoustic signal will sound only when the signal function is activated (see chapter „Setting the acoustic signal“). If the signal function is switched off, no acoustic signal will be heard before or at the end of the measurement.

**NOTE:**

**For maximum reading accuracy in consecutive measurements wait 30 seconds after every 3 to 5 measurements.**

If earwax collects on the measuring probe, this can result in inaccurate temperature measurement or cross-infection between different users.

**Cleaning the measuring probe after each measurement is indispensable.** Therefore the thermometer reminds you to

clean the probe when you switch it off. „CLEAN ME“ appears, and the LED light on the measuring probe flashes for approx. 3 seconds. For instructions on cleaning please refer to the section „Cleaning and disinfection“.

**After cleaning the measuring probe ① with alcohol** wait approx. 5 minutes so that the thermometer reaches its necessary operating temperature before taking another reading.

- 10 short beeps and a red LCD background lighting indicate a patient temperature equal to or exceeding 37.5 °C (99.5 °F).
- Babies and toddlers are best laid on their stomach with the head turned sideways so that the ear faces upwards. When taking a temperature reading on children or grown-ups it is better to stand beside and behind the patient.
- Always measure the temperature in the same ear, as the readings can be different from one ear to the other.
- In the following situations you should take three temperature readings in one ear, and use the highest value as the overall result.
  1. Newborn infants younger than 100 days.
  2. Children younger than three years and with a weak immune system for whom it is vital to know whether or not they have

a temperature. Children developing serious fever symptoms (eg fever convulsions). In cases of high temperatures (above 37.5°C / 99.5 °F in the ear) the patient's temperature must be checked once an hour, and we recommend performing a further check using a pen-type thermometer (rectal measurement).

This helps to better follow temperature variations, eg during the course of a fever.

3. When you are first getting used to using the thermometer it may be useful to take several readings until you get stable values.
4. If the reading does not seem to be in line with the patient's condition, repeat the measurements at shorter intervals.

- Do not measure temperature during or immediately after breastfeeding a baby.
- Do not use the thermometer in an environment with high atmospheric humidity.
- Before and during measurement the patient must not drink, eat or take exercise.
- For newborn infants, physicians recommend only rectal measurement during the first 6 months, as all other measurements might yield contradictory results.

Even the body temperature of a healthy person changes during the course of the day and depends on the activities engaged in. It is highest in the evening and lowest about one hour before waking up.

**Measurements taken on different body parts cannot be compared directly, as the normal human body temperature varies depending on the place of measurement and the time of the day.**

The **normal body temperature range** of a healthy person (see Table) will vary depending on the measurement method, and is subject to slight variations from one person to another. Thus there is no generally accepted “normal” temperature.

- axillary (under the armpit): 34.7 - 37.3 °C (94.4 - 99.1 °F)
- oral (under the tongue): 35.5 - 37.7 °C (95.9 - 99.8 °F)
- rectal (in the anus): 36.6 - 38.0 °C (97.8 - 100.4 °F)
- ear: 35.6 - 37.8 °C (96.0 - 100.0 °F)
- forehead: 35.4 - 37.4 °C (95.7 - 99.3 °F)

## 2.4 Retrieving the 30 readings stored

This thermometer allows to retrieve the last 30 measuring values with date and time information.

**Memory mode:** To access the memory mode, press the M button  when the thermometer is switched off. The memory symbol "M" flashes.

**Retrieval 1 - the last measurement:** To display the last reading taken, press the M button  briefly. The display shows "1" and the memory symbol «M».

**Retrieval 30 - readings stored in the sequence they were measured:** Press the M button  and hold it down to display the last 30 readings one by one.

When you press the M button  again after you have viewed the 30 readings, this sequence will re-start, ie with reading 1.

### Erasing the memory:

1. To access the memory mode, press the M button ⑥ briefly when the thermometer is switched off. The memory symbol «M» blinks.
2. To display the last reading taken, press the M button ⑥ again. The display shows "1" and the memory symbol «M». Then the stored reading appears.
3. In order to clear all stored readings, hold down the "START" button for approx. 2 to 3 seconds. "CLr" appears on the display. Now press the "START" button again.
4. The display lights up green and "PASS" appears. Now all readings are deleted and the memory is reset. Note: It is not possible to clear individual readings.

## 2.5 Error messages and troubleshooting

During use

### The temperature measured is too high:

«H» is displayed, the temperature measured exceeds 100.0 °C (212 °F).

### The temperature measured is too low:


«L» is displayed, the temperature measured is below 0 °C (32 °F).

### Ambient temperature too high:

«H» and «▲» is displayed if the ambient temperature exceeds 40.0 °C (104 °F).

### Ambient temperature too low:

«L» and «▲» is displayed if the ambient temperature is lower than 10.0 °C (50 °F).

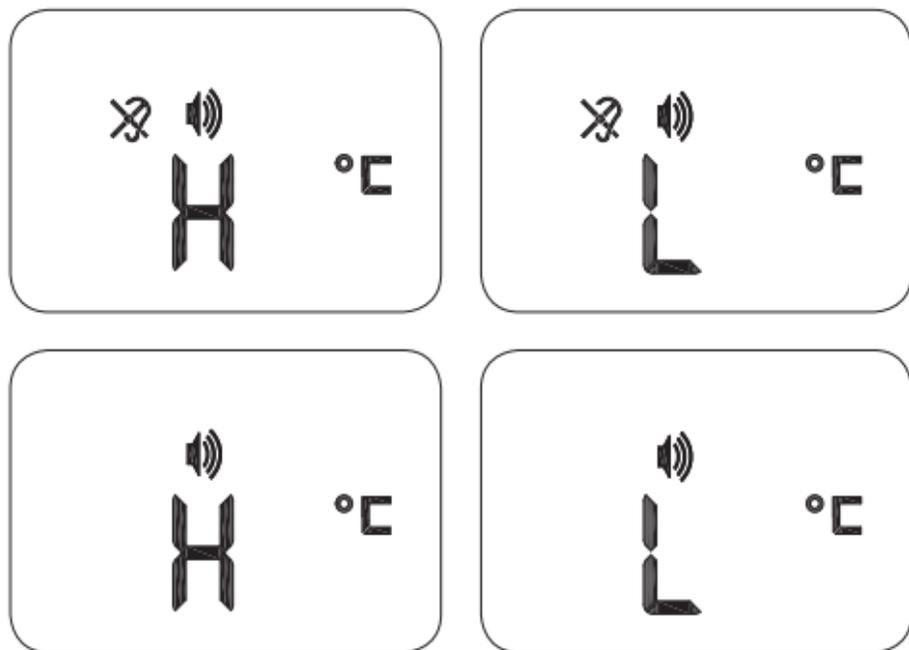**Malfunction signal:**

There is a malfunction or system error.

**Empty display:**

Check whether the battery has been inserted correctly.  
Check the battery polarity ("+" and "-").

**Battery flat:**

Replace the battery without delay if the «▼» symbol is the only symbol lighting up.

### 3.1 Cleaning and disinfection

After each measurement the thermometer reminds you to clean the measuring probe.


“CLEAN ME” appears, and the LED light on the measuring probe flashes for about 3 seconds.

Use an alcohol sponge or a cotton cloth moistened with alcohol (70 % isopropanol) to clean the thermometer housing and the measuring probe. Please make sure that no liquid enters the thermometer. Do not use aggressive cleaners, solvents or benzene for cleaning. Do not immerse the thermometer in water or other cleaning liquids. Take care not to scratch the surface of the measuring probe and of the display.

## 3.2 Battery change

This thermometer operates with a lithium battery type CR2032. Replace the battery if the <<▼>> is the only symbol shown on the display. Remove the battery compartment cover by pushing it in the direction indicated. Insert a new battery into the battery compartment with the + side facing upwards.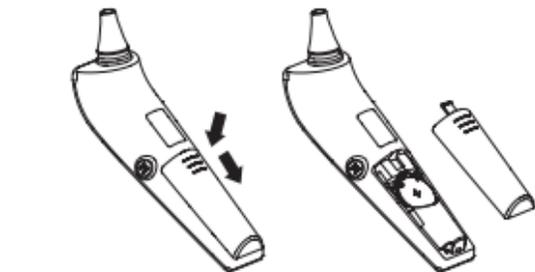

Batteries and electronic devices must not be disposed of as normal domestic trash. They must be disposed of in accordance with the applicable local provisions.

### **WARNING Battery safety information and instructions**

- Do not take the batteries apart!
- If necessary, clean the battery and thermometer contacts before inserting the batteries.
- Remove flat batteries from the thermometer immediately.
- Increased risk of leakage. Avoid contact with skin, eyes and mucous membranes. If you come into contact with battery acid, rinse the skin or mucous membrane with large amounts of water at once and consult a physician immediately.
- In case of swallowing a battery, consult a physician without delay!
- Replace all batteries at the same time.

- Insert only one type of battery. Do not mix different battery types or new and used batteries.
- Insert the batteries correctly, observe the battery polarity.
- Insert the batteries correctly, observe the battery polarity.
- Remove the battery if the thermometer is not to be used for a long period of time.
- Keep the batteries out of reach of children.
- Do not recharge the batteries. Explosion hazard!
- Do not short-circuit the batteries. Explosion hazard!
- Do not throw the thermometer into open fire. Explosion hazard!
- Store new batteries in their packaging and away from metallic objects to avoid short-circuit.
- Do not dispose of used batteries and accumulators with domestic trash, but with hazardous waste, or take them to a battery collection point.

## 4. Technical data

**Typ:** aponorm® ear thermometer COMFORT 3

**Measuring range:** 0 - 100.0 °C (32 - 212 °F)

**Measuring resolution:** 0.1 °C (0.18 °F)

**Measuring accuracy:**

Laboratory: +/- 0.2 °C (0.36 °F), 32,0 - 42,2 °C (89.6 - 107.9 °F)

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**Display:**

- The thermometer is switched on and ready for measurement:  
1 beep
- End of measurement:  
1 long acoustic signal

- System error or malfunction:  
3 beeps
- Fever warning 10 beeps

**Memory:**

- 30 readings with date and time information can be retrieved from the memory.

**Background lighting:**

- The display lights up GREEN for 1 seconds when the thermometer is switched on.
- The display lights up GREEN for 5 seconds if the temperature measured is below 37.5 °C (99.4 °F).
- The display lights up RED for 5 seconds if the temperature measured is equal to or exceeds 37.5 °C (99.4 °F).

**Operating conditions:**

10 – 40 °C (50 - 104 °F)

15 to 95% relative maximum air humidity

**Storage conditions:**

-25 to +55 °C (-13 to +131 °F)

15 to 95% relative maximum air humidity

**Automatic switch-off:**

approx. 1 minute after the last measurement.

**Battery:**

1 x CR2032 battery 3V

**Battery lifetime:**

approx. 1000 measurements (using a new battery)

**Dimensions:** 139 x 39 x 42 mm

**Weight:** 54.5 g (including battery), 51 g (without battery)

**IP Class:** IP22

**Expected service life:** 5 years or 12000 measurements

**Reference to standards:**

EN 12470-5; ASTM E1965; IEC 60601-1;

IEC 60601-1-2 (EMC); IEC 60601-1-11

This thermometer complies with the requirements of the Medical Device Directive 93/42/EEC.

Technical alterations reserved.

## Warranty

This thermometer is covered by a **guarantee for 5 years** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the pharmacy (see last page) confirming the date of purchase, or the till receipt.

The manufacturer will not reimburse the costs for returning the thermometer. Warranty claims submitted by customers residing outside Germany can only be handled if these customers pay the cost of return shipment of the thermometer in advance. Any statutory warranty provisions remain unaffected.

- The warranty covers only the thermometer. Battery and packaging are not covered by the warranty.
- Opening of or modification to the thermometer by the user voids all warranty claims.

- The warranty does not cover damage caused by improper handling, battery leakage, accidents or failure to observe the instructions for use.

## 5 years' warranty

In case of a warranty claim please contact your pharmacy or the place where you bought the thermometer.

## Notes:

- Subject to technical changes without notice.

## Disposal

Electronic devices must not be disposed of as normal domestic trash. They must be disposed of in accordance with the applicable local provisions.

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Microlife AG
WEPA Apothekenbedarf GmbH & Co KG