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Stirn-Thermometer

CONTACT FREE 3

DE

EN

TR

RU

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3 sec

technology by
microlife

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Introduction

Dear customer,

Congratulations on purchasing this aponorm® by microlife forehead thermometer CONTACT-FREE 3.

This thermometer is a quality device for regularly checking and measuring temperature. It is clinically tested and conforms to international standards. Every time the thermometer is switched on, it performs a self-test in order to ensure the specified measuring accuracy at all times. Thanks to its unique technology, the thermometer gives stable readings over and over again. Non-contact measurement facilitates the monitoring of body temperature particularly for infants.

These instructions describe all functions of your thermometer. They include important safety information and instructions for the use of the thermometer. **Please read all information in**

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the operating instructions thoroughly before using the thermometer.

Please retain the guarantee card and these instructions for further reference.

EN

If you have any questions or problems, please contact your local pharmacy. Your pharmacist is the person to speak to for all questions concerning the thermometer. For more detailed information on our products please refer to www.aponorm.de.

We wish you good health at all times!

aponorm® – Your pharmacy brand

Read the instructions carefully before using this device.


Type BF applied part

Important! Please observe the information in this booklet in order to prevent damages to the blood pressure monitor and avoid error messages.

The thermometer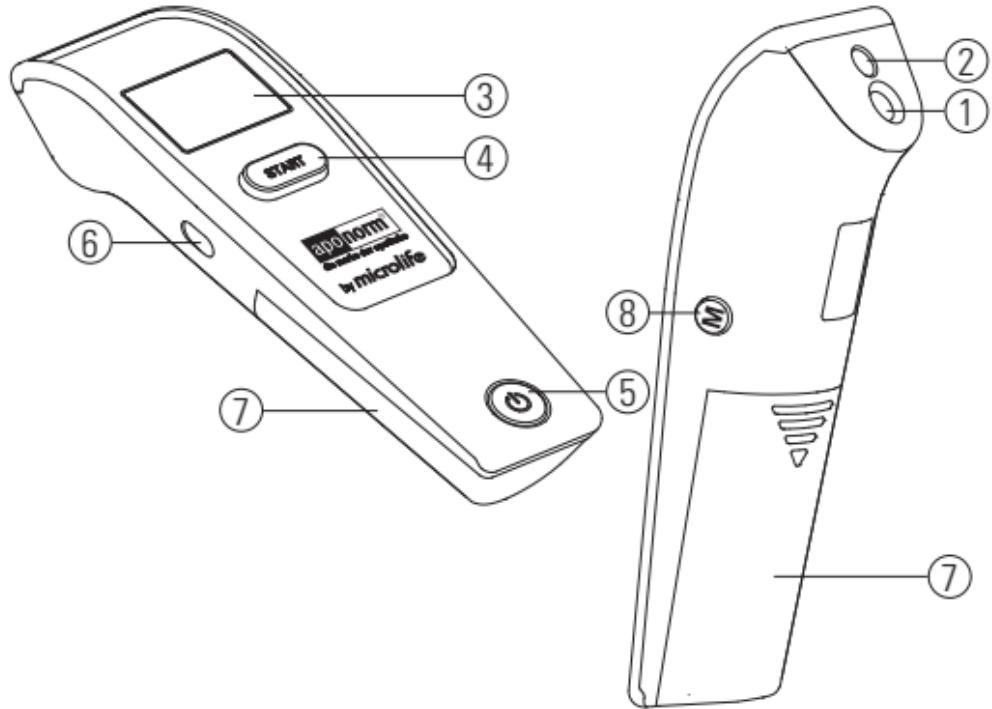

Thermometer:

- ① Measuring probe
- ② Indicator light
- ③ Display
- ④ START button
- ⑤ ON/OFF button
- ⑥ Mode switch
- ⑦ Battery compartment cover
- ⑧ M button (memory)

Table of Contents

1. Before use

- 1.1 Safety information and instructions p. 38
- 1.2 Contents and packaging p. 40
- 1.3 Thermometer features p. 41

2. During use

- 2.1 Display symbols p. 43
- 2.2 Setting of date, time and acoustic signal p. 45
- 2.3 Switching between the body and object modes p. 49
- 2.4 Use p. 50
- 2.5 Retrieving the last readings stored p. 54
- 2.6 Error messages and troubleshooting p. 57

3. After use

- 3.1 Cleaning and disinfection p. 60
- 3.2 Battery change p. 60

4. Technical data

p. 63

EN

1.1 Safety information and instructions

Safety

This unit is designed for private use only.

- The patient and the thermometer must be in the same environment having a constant temperature for at least 30 minutes before a reading can be taken. (Store the thermometer where you usually take measurements.)
- This thermometer must only be used for the purposes described in these instructions. The manufacturer will not accept any liability for damage caused by improper use.
- **The thermometer is not waterproof. Do not immerse it in water or other liquids. For cleaning and disinfection please observe the instructions in section "Cleaning and disinfection".**

- Do not use this thermometer if you think it is damaged or if you notice anything unusual.
- Open the battery compartment cover only for battery replacement (see instructions).
- The temperature of the forehead and temples is different from the **body's core temperature** (rectal/oral). Often patients having a temperature develop sweat on their forehead. This is what we call the "skin cooling effect". In this case temperature measurements can be lower than expected. If the reading measured does not correspond to the patient's condition, please repeat the measurement every 15 minutes. This helps to faster recognize temperature variations, eg during the course of a fever.

Keep dry.

- This thermometer contains sensitive components and must be handled carefully. Observe the storage and operating conditions described in the "Technical data" section.
- In case of malfunction or damage do not repair the thermometer yourself. This will void all warranty claims. Please contact the pharmacy where you bought the device. Your pharmacist will help you with all questions concerning the thermometer.
- Please ensure that unsupervised children do not use this thermometer; some parts are small enough to be swallowed.
- The function of this device may be compromised when used close to strong electromagnetic fields such as mobile phones or radio installations and we recommend a distance of at least 1 m. In cases where you suspect this to be unavoidable, please verify if the device is working properly before use.

- Do not use the thermometer in a humid environment.
- If the thermometer is not going to be used for a prolonged period the batteries should be removed.

Protect the thermometer from:

- extreme temperatures and/or temperature variations
- impact and falls
- contamination and dust
- direct sunlight

**WARNING:**

Using the thermometer is not an alternative to consulting a physician.

1.2 Contents and packaging

Please check first that the thermometer is complete and not damaged. If in doubt do not use the thermometer and return it to your pharmacy.

Contents:

- 1 **aponorm®** forehead thermometer CONTACT-FREE 3
- 2 x 1.5 V batteries; size AAA (batteries already inserted)
- 1 storage case
- 1 set of instructions for use

All packaging is recyclable and can be fed back into the raw materials cycle. Please ensure proper disposal of any packaging material no longer required. If you find any transport damage on unpacking your thermometer, please contact your pharmacy immediately.

WARNING:

Keep packaging film away from babies and young children. Risk of suffocation!

1.3 Thermometer features

Before use

Quick measurement

The innovative infrared technology allows measurements without the thermometer even touching the object. When handled properly, the thermometer makes reliable and hygienic measurements within just seconds.

Measuring of body and surface temperatures possible

The thermometer has an expanded measuring range from 0 °C - 100 °C (32.0 °F - 212.0 °F). This means that it can be used for **measuring the body temperature** (body mode) as well as for **measuring surface temperatures** (object mode), eg for the following objects:

- Surface temperature of milk in baby feeding bottles
- Ambient temperature

CAUTION:

Surface temperature is different from core temperature. For measuring the core temperature of liquids use a suitable waterproof thermometer.

EN

Accurate and reliable

The unique measuring probe with modern infrared sensor ensures that all readings are accurate and reliable when the thermometer is used properly.

Convenient and simple handling

- The ergonomic design allows convenient and simple handling and use of the thermometer.
- You can even measure the temperature of a sleeping child without disturbing them (see information on the acoustic signal).
- The thermometer is very fast and therefore convenient, particularly for children or bedridden patients.

Retrieval of several readings

The last 30 readings are stored and can be retrieved in the memory mode. This helps to better follow temperature variations, eg during the course of a fever.

Fever warning

10 short beeps and a red LCD background lighting indicate a patient temperature exceeding 37.5 °C (99.3 °F).

Reliable and hygienic

- No direct contact with the skin
- Suitable for children from 24 months

2.1 Display symbols


During use

List of all display symbols:

Press the ON/OFF button ⑤ to switch the thermometer on:
All display symbols are visible for a period of 1 second.

Ready for use:


The thermometer is ready for use when the “°C” or “°F” symbol flashes while the mode symbol (body or object) is displayed permanently.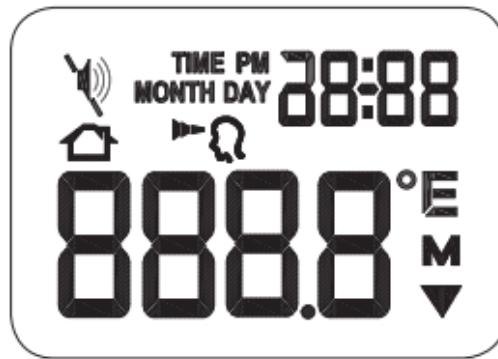

EN


Measurement finished:

The reading value appears on the display with the “°C” or “°F” symbol and the mode symbol. When the “°C” or “°F” symbol flashes again, the thermometer is ready for taking another reading.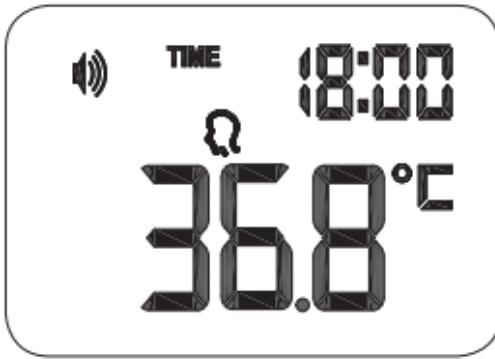

Battery low:

When the thermometer is switched on, the battery symbol «▼» lights up permanently to remind the user that the batteries must be replaced.

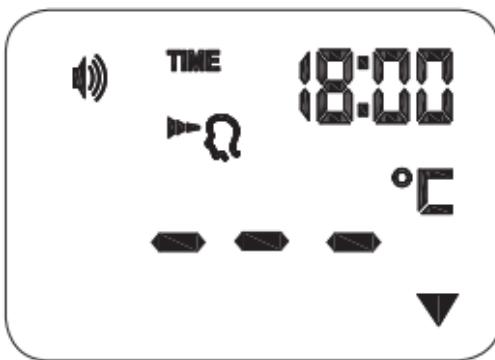2.2 Setting of date, time and acoustic signal

During use

Setting of time and date

1. When using the thermometer for the first time or when new batteries have been inserted, the year flashes in the display. Press the START button ④ to set the correct year. Press the M button ⑧ for confirmation and for changing to the month setting mode.
2. Now the correct month can be set by pressing the START button ④. Press the M button ⑧ again for confirmation and for changing to the date setting mode.

EN

3. Follow the instructions above to set the date, choose the 12 or 24 hour clock and set the hours and minutes.
4. Once you have set the minutes and pressed the M button ⑧, the date and time are set, and the time is displayed.

If you do not press any other button within 20 seconds, the thermometer will automatically revert to "Ready for use" mode.

To interrupt the time setting:

Press the ON/OFF button ⑤ when setting the time. The display will show the date/time as “--:--”. Then press the ON/OFF button ⑤ in order to start temperature measurement. If you do not press any other button within 20 seconds, the thermometer will automatically switch off.

Changing the present date and time settings:

Keep the M button ⑧ pressed (when the thermometer is switched off) for approx. 3 seconds until the year indication flashes. Now you can enter the new values as described above.

Setting the acoustic signal

1. When the thermometer is switched off, keep the ON/OFF button ⑤ pressed for about 5 seconds in order to set the acoustic signal.
2. Press the ON/OFF button ⑤ again in order to switch the acoustic signal either on or off. This acoustic signal is activated if the acoustic signal icon is not crossed out.
If you do not press any other button within 5 seconds, the thermometer will automatically revert to the "Ready for use" mode.

2.3 Switching between the body and object modes

During use

In order to change from body mode to object mode, push the mode switch ⑥ on the side of the thermometer down towards the . To return to the body mode, push the switch up again towards the .

EN

2.4 Use

Temperature measurement in the body mode

1. **Press the** ON/OFF button ⑤. The display lights up and shows all display symbols for a period of 1 second.
2. **The thermometer** is ready for use as soon as the “°C” or “°F” symbol flashes and a beep is heard.
3. **Point** the thermometer to the middle of the patient's forehead at a distance of 1 to 5 cm. If the forehead is covered with hair or in sweat or dirt, clean it first in order to ensure accurate measurement.
4. **Press** the START button ④ once and move the thermometer evenly from the middle of the forehead to the temples (about 1 cm above the eyebrow).The blue indicator light indicates the measuring range.

After 3 seconds a long acoustic signal is heard confirming the end of the measurement. If you did not reach the temples before the acoustic signal sounded, repeat the measurement as described above, but move the thermometer a bit faster.

5. **Read** the temperature measured on the display.

Important information about the acoustic signal: This acoustic signal will sound only when the signal function is activated (**see chapter “Setting the acoustic signal”**). If the signal function is switched off, no acoustic signal will be heard before or at the end of the measurement.

Temperature measurement in the object mode

1. **Follow** steps 1 to 3 as described above, then point the thermometer to the centre of the object the temperature of which you want to measure, at a distance of 1 to 5 cm from its surface. **Press** the ON/OFF button ④. After 3 seconds a long acoustic signal is heard confirming the end of the measurement.
2. **Read** the temperature shown on the display.

NOTE:

- The patient and the thermometer must be in the same environment having a constant temperature for at least 30 minutes before a reading can be taken. (Store the thermometer where you usually take measurements.)

- Do not measure temperature during or immediately after breastfeeding a baby.
- For newborn infants and toddlers, physicians recommend only rectal measurement (measurement of the core temperature) during the first 24 months, as all other measurements might yield contradictory results. When you use a contact-free thermometer on newborn infants and toddlers, we recommend verifying the result with a rectal measurement in each case.
- Do not use the thermometer in an environment with high atmospheric humidity.
- Before and during measurement the patient must not drink, eat or take exercise.

- During measuring the patient's head must remain still. Head movements can influence the reading, as the measuring points above the eyebrow towards the temple cannot be recorded correctly if the patient moves.
- Do not remove the thermometer from the measuring area (body or object) before the acoustic end signal is heard.
- Even if the thermometer is not in contact with the patient's skin, clean the probe thoroughly with an alcohol swab and wait for 15 minutes before measuring another person's temperature. Due to external influences the lens can become covered with dust.
- 10 short beeps and a red LCD background lighting indicate a patient temperature exceeding 37.5 °C (99.3 °F).

Note:

In the following situations you should take three temperature readings when measuring the temperature on the forehead, and use the highest value as the overall result.

1. Children younger than three and with a weak immune system for whom it is vital to know whether or not they have a temperature. Children developing serious fever symptoms (eg fever convulsions). In cases of high temperatures (above 37.5°C / 99.3°F on the forehead) the patient's temperature must be checked once an hour, and we recommend performing a further check using a pen-type thermometer (rectal measurement).

This helps to better follow temperature variations, eg during the course of a fever.

2. When you are first getting used to using the thermometer it may be useful to take several readings until you get stable values.
3. If the reading does not seem to be in line with the patient's condition, repeat the measurements at shorter intervals.

Even the body temperature of a healthy person changes during the course of the day and depends on the activities engaged in. It is highest in the evening and lowest about one hour before waking up.

Measurements taken on different body parts cannot be compared directly, as the normal human body temperature varies depending on the place of measurement and the time of the day.


The **normal body temperature range** of a healthy person (see Table) will vary depending on the measurement method, and is subject to slight variations from one person to another. Thus there is no generally accepted "normal" temperature.

- axillary (under the armpit): 34.7 - 37.3 °C (95.7 - 99.3 °F)
- oral (under the tongue): 35.5 - 37.7 °C (95.7 - 99.3 °F)
- rectal (in the anus): 36.6 - 38.0 °C (95.7 - 99.3 °F)
- forehead (at a distance of 1 to 5 cm): 35.4 - 37.4 °C (95.7 - 99.3 °F)
- ear: 35.6 - 37.8 °C (96.1 - 100.0 °F)

2.5 Retrieving the last readings stored

The thermometer automatically stores the last 30 readings. You can retrieve the last 30 values along with their date and time from the memory.

Memory mode: To access the memory mode, press the M button ⑧ briefly when the thermometer is switched off. The memory symbol «M» flashes.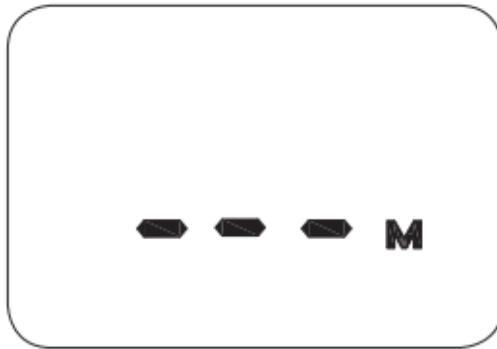


Retrieval 1 - the last measurement: To display the last reading taken, press the M button ⑧ again. The display shows "1" and the memory symbol M. Then the stored value is shown, its date and time appear alternately.

Retrieval 30 - readings stored in the sequence they were measured: Press the M button ⑧ and hold it down to display the last 30 readings one by one.

The last reading taken (the latest value) is in position 1.

The first reading taken (the oldest value) is in position 30.

When you press the M button ⑧ again after you have viewed the 30 readings, this sequence will re-start, ie with reading 1.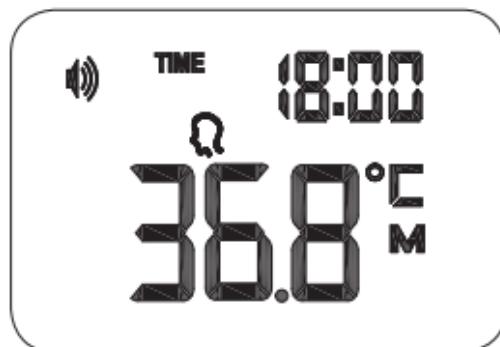

Erasing the memory:

1. To access the memory mode, press the M button ⑧ briefly when the thermometer is switched off.
The memory symbol «**M**» flashes.
2. To display the last reading taken, press the M button ⑧ again. The display shows “1” and the memory symbol «**M**». Then the stored reading appears.
3. In order to clear all stored readings, hold down the “START” button for approx. 2 to 3 seconds.
“CLr” appears on the display. Now press the “START” button again.
4. The display lights up green and “PASS” appears.
Now all readings are deleted and the memory is reset.
Note: It is not possible to clear individual readings.

2.6 Error messages and troubleshooting



During use

The temperature measured is too high:

«H» is displayed if the temperature measured exceeds 42.2 °C (107.9 °F) in the body mode or 100 °C (212 °F) in the object mode.

The temperature measured is too low:

«L» is displayed if the temperature measured is lower than 34.0 °C (93.2 °F) in the body mode or 0 °C (32 °F) in the object mode.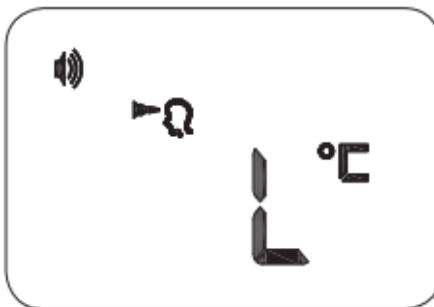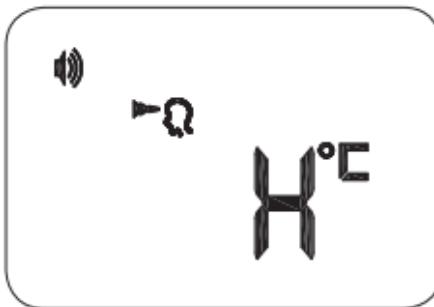


EN

Ambient temperature too high:

At excessive ambient temperatures measurement is not possible.

«H» and  are displayed if the ambient temperature exceeds 40.0 °C (104.0 °F), independently of the measuring mode you are in.

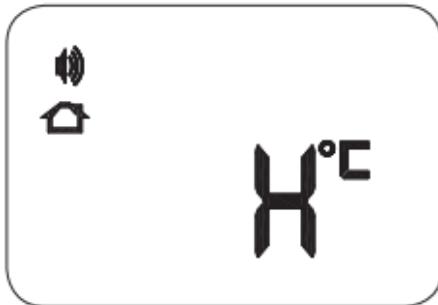Ambient temperature too low:

At too low ambient temperatures measurement is not possible.

«L» and  are displayed if the temperature measured is lower than 16.0 °C (60.8 °F) in the body mode or 5.0 °C (41 °F) in the object mode.

Malfunction signal:

There is a malfunction or system error.

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
Empty display:

Check whether the batteries have been inserted correctly.

Check the battery polarity (<+> and <->).

Batteries flat:

Replace the batteries without delay if the «▼» symbol is the only symbol lighting up.


3.1 Cleaning and disinfection

Use an alcohol wipe or a cotton cloth moistened with alcohol (70% isopropanol) to clean the thermometer housing and the measuring probe. Please make sure that no liquid enters the thermometer. Do not use aggressive cleaners, solvents or benzene for cleaning. Do not immerse the thermometer in water or other cleaning liquids. Take care not to scratch the surface of the measuring probe and of the display.

3.2 Battery change

The thermometer is supplied with two new long-life 1.5 V batteries AAA. The batteries must be replaced if the «▼» is the only symbol shown on the display. Remove the battery compartment cover ⑦, pushing it in the direction indicated. Replace the batteries - ensure correct polarity as shown by the symbols in the compartment.

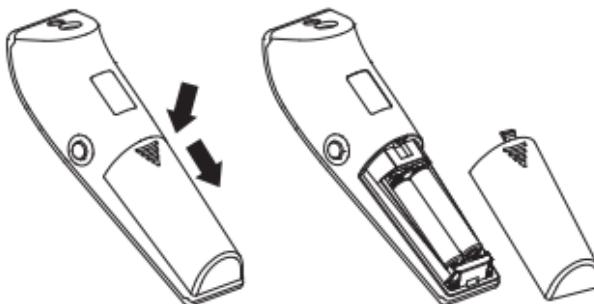WARNING**Battery safety information and instructions**

- Do not take the batteries apart!
- If necessary, clean the battery and thermometer contacts before inserting the batteries.
- Remove flat batteries from the thermometer immediately.
- Increased risk of leakage. Avoid contact with skin, eyes and mucous membranes. If you come into contact with battery acid, rinse the skin or mucous membrane with large amounts of water at once and consult a physician immediately.
- In case of swallowing a battery, consult a physician without delay!

- Replace all batteries at the same time.
- Insert only one type of battery. Do not mix different battery types or new and used batteries.
- Insert the batteries correctly, observe the battery polarity.
- Keep the battery compartment closed at all times.
- Remove the batteries if the thermometer is not to be used for a long period of time.
- Keep the batteries out of reach of children.

- Do not recharge the batteries. Explosion hazard!
- Do not short-circuit the batteries. Explosion hazard!
- Do not throw the thermometer into open fire. Explosion hazard!
- Store new batteries in their packaging and away from metallic objects to avoid short-circuit.
- Do not dispose of used batteries and accumulators with domestic trash, but with hazardous waste, or take them to a battery collection point.

4. Technical data

Type: aponorm® forehead thermometer CONTACT-FREE 3

Measuring range:

Body mode: 34.0 - 42.2 °C (93.2 - 107.9 °F)

Object mode: 0 - 100.0 °C (32 - 212 °F)

Measuring resolution: 0.1 °C (0.18 °F)

Measuring accuracy:

+/- 0.2 °C (0.36 °F), 36.0 - 39.0 °C (96.8 - 102.2 °F)

(the measuring accuracy is +/- 0.2 °C (+/- 0.36 °F) for temperatures between 36.0 °C and 39.0 °C (96.8 and 102.2 °F))

Display: Liquid-crystal display (LCD)

Acoustic alarms:

- The thermometer is switched on and ready for measurement:
1 beep

- Conclusion of measurement: 1 long acoustic signal (1 s) if the reading is below 37.5 °C (99.4 °F), 10 beeps if the reading is equal to or exceeds 37.5 °C (99.4 °F).
- System error or malfunction: 3 beeps

EN

Memory:

- 30 readings with date and time information can be retrieved from the memory.

Background lighting:

- The display lights up GREEN for 4 seconds when the thermometer is switched on.
- The display lights up GREEN for 5 seconds if the temperature measured is below 37.5 °C (99.4 °F).
- The display lights up RED for 5 seconds if the temperature measured is equal to or exceeds 37.5 °C (99.4 °F).

Operating conditions (ambient temperature):

Body mode: 16 - 40.0 °C (60.8 - 104 °F)

Object mode: 5 - 40.0 °C (41 - 104 °F)

Storage conditions:

-20 °C to +50 °C (-4 to 122 °F)

15 to 95% relative maximum air humidity

Automatic switch-off:

approx. 1 minute after the last measurement.

Batteries: 2 x 1.5 V alkaline batteries; size AAA

Battery lifetime:

approx. 2000 measurements (using new batteries)

Size: 141.1 x 43.3 x 36.9 mm

Weight: 90 g (including batteries), 67 g (without batteries)

IP Class: IP21

Expected service life: 5 years or 12000 measurements

Reference to standards:

ASTM E1965; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11

This thermometer complies with the requirements of the Medical Device Directive 93/42/EEC. Technical alterations reserved.

Warranty

This thermometer is covered by a **guarantee for 5 years** from the date of purchase. The guarantee is valid only on presentation of the guarantee card completed by the pharmacy (see last page) confirming the date of purchase, or the till receipt.

The manufacturer will not reimburse the costs for returning the thermometer. Warranty claims submitted by customers residing outside Germany can only be handled if these customers pay the cost of return shipment of the thermometer in advance. Any statutory warranty provisions remain unaffected.

- The warranty covers only the thermometer. Batteries and packaging are not covered by the warranty.
- Opening of or modification to the thermometer by the user voids all warranty claims.

- The warranty does not cover damage caused by improper handling, battery leakage, accidents or failure to observe the instructions for use.

5 years' warranty

In case of a warranty claim please contact your pharmacy or the place where you bought the thermometer.

Notes:

- Subject to technical changes without notice.

Disposal:

Electronic devices must not be disposed of as normal domestic trash. They must be disposed of in accordance with the applicable local provisions.

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Microlife AG
WEPA Apothekenbedarf GmbH & Co KG